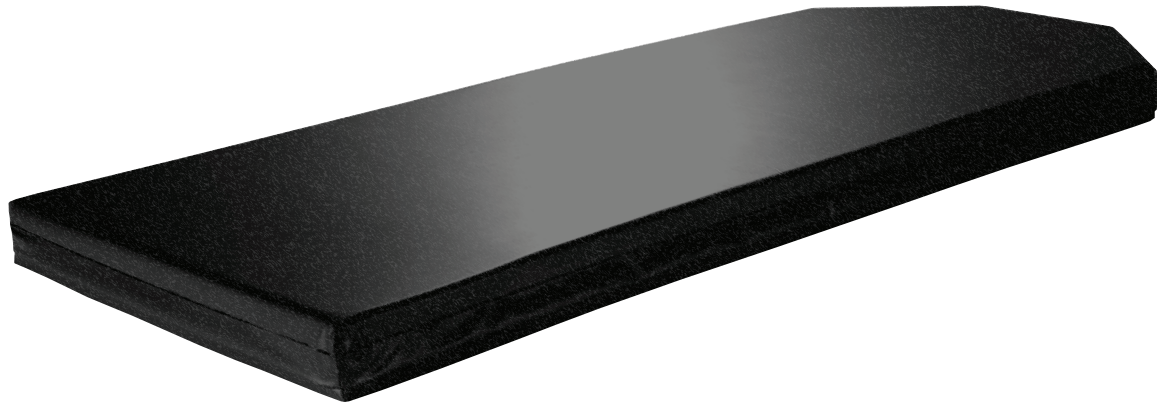
# ST1™ and ST1X™ Foam Mattress

Operations Manual

**REF** 6300-0-100

EN  
CS  
DA  
DE  
EL  
ES  
ET  
FI

FR  
HR  
HU  
IT  
KO  
LV  
NL  
NO

PL  
PT  
RO  
RU  
SK  
SL  
SV  
ZH

# Symbols

	Refer to instruction manual/booklet
	Operating instructions/Consult instructions for use
	General warning
	Caution
	Catalogue number
	Serial number
	CE mark
	Manufacturer (XXXX indicates year of manufacture)
	Safe working load
	Type B applied part
	Wash by hand
	Do not tumble dry
	Do not dry clean
	Do not iron
	Allow to completely air dry
	Chlorinated bleach

# Table of Contents

- Symbols ..... 1
  - Warning/Caution/Note Definition ..... 2
    - Summary of safety precautions ..... 2
- Setup the mattress ..... 3
- Introduction ..... 4
  - Product description ..... 4
  - Intended use ..... 4
    - Product not intended for use ..... 4
  - Expected life ..... 5
  - Contraindications ..... 5
  - Specifications ..... 5
  - Contact information ..... 5
  - Serial number location ..... 6
- Preventive maintenance ..... 7
- Cleaning ..... 8
- Disinfecting ..... 9

# Warning/Caution/Note Definition

The words **WARNING**, **CAUTION**, and **NOTE** carry special meanings and should be carefully reviewed.

---

**WARNING** - Alerts the reader about a situation which, if not avoided, could result in death or serious injury. It may also describe potential serious adverse reactions and safety hazards.

---

**CAUTION** - Alerts the reader of a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the product or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur as a result of use or misuse.

---

**Note** - Provides special information to make maintenance easier or important instructions clearer.

## Summary of safety precautions

Always read and strictly follow the warnings and cautions listed on this page. Service only by qualified personnel.

---

### WARNING

- Always use linens with the mattress.
  - Do not stick needles into the mattress cover. Holes may allow body fluids to enter the inside (inner core) of the mattress and may cause cross-contamination or product damage.
  - Always use the mattress with a compatible frame as indicated in the specification section of this manual.
  - Do not wash the internal components of this mattress. Discard the mattress if contamination is found inside.
  - Do not immerse the mattress in cleaning or disinfectant solutions.
  - Do not allow liquid to pool on the mattress.
  - Do not iron, dry clean, or tumble dry the mattress cover.
  - Always disinfect the mattress following your hospital protocols to avoid the risk of cross-contamination and infection.
  - Do not use **Virex® TB** to disinfect this product.
  - Do not use accelerated hydrogen peroxides or quaternaries that contain glycol ethers as they may damage the mattress cover.
  - Always inspect the mattress each time you clean the mattress cover. Follow your hospital protocols and complete preventive maintenance each time you clean the mattress cover. If compromised, remove the mattress from use and replace the product to prevent cross-contamination.
- 

### CAUTION

- Improper usage of the product can cause injury to the patient or operator. Operate the product only as described in this manual.
  - Do not modify the product or any components of the product. Modifying the product can cause unpredictable operation resulting in injury to patient or operator. Modifying the product also voids its warranty.
  - Always use the 6300-1-000 mattress cover on the foam core.
  - Always make sure that you wipe the product with clean water. Dry each product after cleaning. Some cleaning agents are corrosive in nature and may cause damage to the product. Failure to follow these cleaning instructions may void your warranty.
  - Do not use cleaning agents and disinfectants with aggressive chemicals as they will reduce the expected life of the mattress cover.
  - Do not allow liquid to seep into the zipper area or watershed cover barrier when you clean the mattress bottom. Fluids allowed to come in contact with the zipper may leak into the mattress.
  - Always dry the mattress cover before you store, add linens, or place a patient on the surface. A dry product helps to prevent impaired product performance.
  - Do not overexpose the mattress cover to high concentrations of disinfectant solutions as they may degrade the mattress cover.
  - Failure to follow the manufacturing cleaning instructions and Stryker operational instructions may affect useful life of the mattress.
-

# Setup the mattress

---

## **WARNING**

- Always use linens with the mattress.
  - Do not stick needles into the mattress cover. Holes may allow body fluids to enter the inside (inner core) of the mattress and may cause cross-contamination or product damage.
  - Always use the mattress with a compatible frame as indicated in the specification section of this manual.
- 

To setup the mattress:

1. Place the mattress on a compatible stretcher.
2. Make sure that you align the mattress with the Stryker logo at the head end of the stretcher.
3. Align the hook and loop fastener onto the bottom cover of the mattress to the litter deck of the stretcher.
4. Make sure that the water shed flaps cover the zipper.
5. Place linens on the mattress before patient use. Follow your hospital protocols.

# Introduction

This manual assists you with the operation or maintenance of your Stryker product. Read this manual before operating or maintaining this product. Set methods and procedures to educate and train your staff on the safe operation or maintenance of this product.

---

## CAUTION

- Improper usage of the product can cause injury to the patient or operator. Operate the product only as described in this manual.
  - Do not modify the product or any components of the product. Modifying the product can cause unpredictable operation resulting in injury to patient or operator. Modifying the product also voids its warranty.
- 

## Note

- This manual is a permanent part of the product and should remain with the product even if the product is sold.
- Stryker continually seeks advancements in product design and quality. This manual contains the most current product information available at the time of printing. There may be minor discrepancies between your product and this manual. If you have any questions, contact Stryker Customer Service.

## Product description

The Stryker Model 6300-0-100 **ST1™** and **ST1-X™** foam mattress is a non-powered mattress for use with the Model 6300 **ST1** and **ST1-X** series stretcher.

## Intended use

The **ST1™** and **ST1-X™** foam mattress is for use as a rest or sleep surface for human adult patients in a hospital setting.

This mattress shall be used with a mattress cover at all times. The mattress cover can interact with all external skin.

Operators of this mattress includes healthcare professionals (such as nurses, nurse aids, or doctors).

Stryker promotes the clinical assessment of each patient and appropriate usage by the operator.

## Product not intended for use

- In a home health environment setting
- As a sterile product
- In the presence of flammable anesthetics
- As support for a patient in a prone position
- With patients who have unstable spinal cord injuries
- With an oxygen tent
- For stay longer than 24 hours

## Expected life

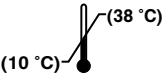
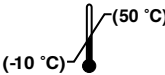
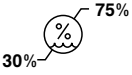
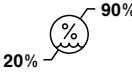
The ST1™ and ST1-X™ foam mattress has a 1 year expected life under normal use, conditions, and with appropriate periodic maintenance.

## Contraindications

None known.

## Specifications

 Safe working load indicates the sum of the patient, mattress, and accessory weight	250 kg
 Maximum patient weight	215 kg
Length	193 cm
Width	62 cm
Thickness	8 cm
Product weight	4,5 ± 1.0 kg
Compatible stretcher frame	6300-000-200, 6300-000-100
Mattress material	
Foam	Polyurethane
Cover	Polyurethane and polyamide coated polyester

Environmental conditions	Operation	Storage and transportation
Ambient temperature		
Relative humidity (non-condensing)		

See mattress label for applicable flammability standards.

Stryker reserves the right to change specifications without notice.

## Contact information

Contact Stryker Customer Service or Technical Support at: +1 800-327-0770.

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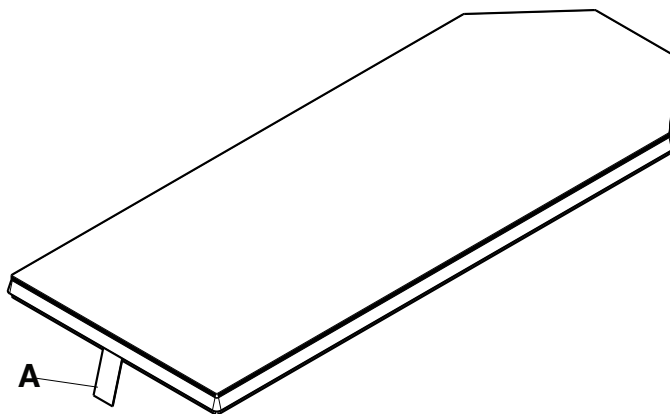
To view your operations or maintenance manual online, see <https://techweb.stryker.com/>.

Have the serial number (A) of your Stryker product available when calling Stryker Customer Service or Technical Support. Include the serial number in all written communication.

## Serial number location

Unzip the mattress cover to locate the product label and serial number.

# Preventive maintenance

Remove product from use before performing preventive maintenance.

---

**WARNING** - Always inspect the mattress each time you clean the mattress cover. Follow your hospital protocols and complete preventive maintenance each time you clean the mattress cover. If compromised, remove the mattress from use and replace the product to prevent cross-contamination.

---

**Note** - Clean and disinfect the exterior of the mattress before inspection, if applicable.

Inspect the following items:

- \_\_\_\_\_ Zipper and cover (top and bottom) are free of tears, cuts, or holes
- \_\_\_\_\_ Unzip the cover to check internal components for signs of stains from fluid ingress or contamination
- \_\_\_\_\_ Foam and other components have not degraded or come apart

Product serial number:
Completed by:
Date:

# Cleaning

Cleaning and disinfecting are two separate processes. Clean before disinfecting to make sure that the cleaning agent is effective.

---

## WARNING

- Do not wash the internal components of this mattress. Discard the mattress if contamination is found inside.
  - Do not immerse the mattress in cleaning or disinfectant solutions.
  - Do not allow liquid to pool on the mattress.
  - Do not iron, dry clean, or tumble dry the mattress cover.
- 

## CAUTION

- Always use the 6300-1-000 mattress cover on the foam core.
  - Always make sure that you wipe the product with clean water. Dry each product after cleaning. Some cleaning agents are corrosive in nature and may cause damage to the product. Failure to follow these cleaning instructions may void your warranty.
  - Do not use cleaning agents and disinfectants with aggressive chemicals as they will reduce the expected life of the mattress cover.
  - Do not allow liquid to seep into the zipper area or watershed cover barrier when you clean the mattress bottom. Fluids allowed to come in contact with the zipper may leak into the mattress.
  - Always dry the mattress cover before you store, add linens, or place a patient on the surface. A dry product helps to prevent impaired product performance.
- 

Always follow hospital protocols for cleaning and disinfecting.

To clean the mattress cover:

1. Wipe the mattress cover with a clean, soft, damp cloth with a mild soap and water solution to remove foreign material.
2. Wipe the mattress cover with a clean, dry cloth to remove excess liquid or cleaning agent.
3. Allow the mattress cover to dry.

**Note** - Laundering is not recommended as it may decrease the useful life of the mattress.

# Disinfecting

---

## WARNING

- Always disinfect the mattress following your hospital protocols to avoid the risk of cross-contamination and infection.
  - Do not immerse the mattress in cleaning or disinfectant solutions.
  - Do not allow liquid to pool on the mattress.
  - Do not use **Virex® TB** to disinfect this product.
  - Do not use accelerated hydrogen peroxides or quaternaries that contain glycol ethers as they may damage the mattress cover.
- 

## CAUTION

- Always make sure that you wipe the product with clean water. Dry each product after cleaning. Some cleaning agents are corrosive in nature and may cause damage to the product. Failure to follow these cleaning instructions may void your warranty.
  - Always dry the mattress cover before you store, add linens, or place a patient on the surface. A dry product helps to prevent impaired product performance.
  - Do not overexpose the mattress cover to high concentrations of disinfectant solutions as they may degrade the mattress cover.
  - Do not allow liquid to seep into the zipper area or watershed cover barrier when you clean the mattress bottom. Fluids allowed to come in contact with the zipper may leak into the mattress.
  - Do not use cleaning agents and disinfectants with aggressive chemicals as they will reduce the expected life of the mattress cover.
  - Failure to follow the manufacturing cleaning instructions and Stryker operational instructions may affect useful life of the mattress.
- 

Recommended disinfectants:

- Quaternaries without glycol ethers - 2100 ppm active
- Chlorinated bleach - 1000 ppm active
- 70% Isopropyl alcohol (700,000 ppm)

Always follow hospital protocols for cleaning and disinfecting.

To disinfect the mattress cover:

1. Clean and dry the mattress cover before you apply disinfectants.
2. Apply recommended disinfectant solution with pre-soaked wipes or damp cloth. Do not soak the mattress.

**Note** - Make sure that you follow the disinfectant manufacturer's instructions for appropriate contact time and rinse requirements.

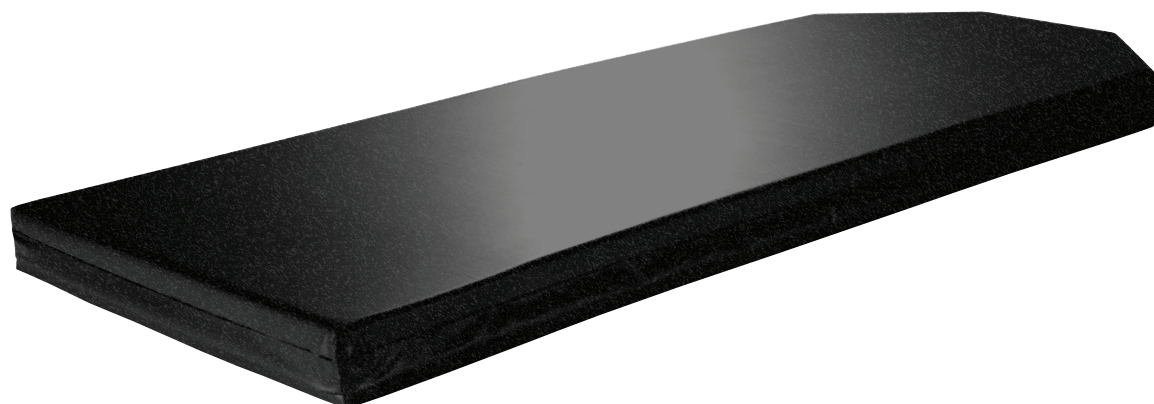
3. Wipe the mattress cover with a clean, dry cloth to remove any excess liquid or disinfectant.
4. Allow the mattress cover to dry.

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Stryker European Operations B.V.